


An easy and quick way to share your blood glucose results with your healthcare professional

Become a CONTOUR® DIABETES app user and connect with your healthcare professional remotely*

1

**Download
CONTOUR®
DIABETES app**

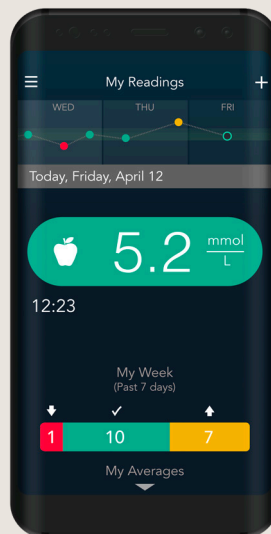

Free download




Download on the
App Store

GET IT ON
Google Play

For a list of compatible devices,
please visit
compatibility.contourone.com

2

**Create
an account**

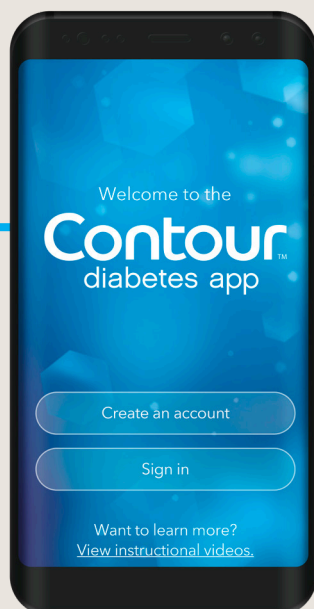

3

**Pair your compatible
CONTOUR® meter**



For more information visit:
diabetes.ascensia.ie

*Patients need to be registered in GlucoContro.online and approve sharing their data with their healthcare professional

Simple onboarding to enable remote blood glucose assessment with your healthcare professional

Wait for the invitation from your healthcare professional

Invitation via email

or

Invitation via QR code

Receive an email invitation and click on the link*



1

Scan the QR code generated during the visit

Add new Patient

Manual QR Code

Mandatory

First Name Last Name

Enter first name Enter last name

Country

Ireland

Optional

I want to send an e-mail invitation to this patient

E-Mail address

Enter E-mail address

Add new Patient

Add new Patient

Manual QR Code

EXAMPLE

2

Log in with your CONTOUR®DIABETES app username and password

Patient Professional

Contour To log in to Your account use Contour Cloud credentials.

E-mail address

Enter e-mail address

Password [Forgot your password?](#)

Enter password

Remember me

Sign in Don't have an account? [Sign up](#)

3

Fill out your information



Now your blood glucose data is accessible for your healthcare professional via GlucoContro.online

*Healthcare professional to initiate the invitation



Ascensia Customer Service Helpline:
Tel: 1890 920111
diabetessupport@ascensia.com
www.diabetes.ascensia.ie